
What your test contains

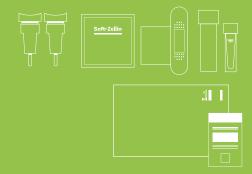

1x activation card with adhesive label

Information about your test

IMPORTANT INFORMATION

For best results, we recommend collecting your sample on Monday to lesday - and post it on the same day it was collected (within 24 hours; please consider any public holidays).

Please note:

We can only analyse your blood sample after you have activated your Test ID and Online Order Reference Code. (Further details can be found in the instructions or at www.lifelabtesting.com/help

IVD In vitro diagnostics

Read the instruction

before taking the sample

(2) For single use only

16 Store at a temperature between 2°C and 30°C

▼1 Contains 1 kit

Manufacturer

LOT Batch Number

REF Order Ref

Expiration Date

GENERAL INFORMATION

- Please read these instructions carefully before you begin
- Please don't take your blood sample immediately after strenuous exercise
- If you have any difficulties taking your blood sample (or if the kit is incomplete), please contact our Customer Service Department. They always treat such matters confidentially
- This is a blood sample kit for self-testing. Please ensure your blood sample is at least 300 uL volume
- We encourage you to return the test kit packaging to us for recycling

CONTAGIOUS ILLNESSES

Do not use this kit if you suffer from a blood-borne contagious illness such as hepatitis or HIV.

WARNINGS

- Store this kit out of reach of children
- This kit is for the use of people aged over 5 years; under 16s should be supervised by an
- If you're pregnant or breastfeeding, please consult your doctor before using this kit The lancets are exclusively for your own use.
- If you suffer from a congenital or acquired blood-clotting disorder or are taking anticoagulant medication, please consult your doctor before using this kit
- The included lancets are exclusively for your own use

If you have any questions, please send us an email at help@lifelabtesting.com or visit our website to find out more: www.lifelabtesting.com/help

STEP 1

STEP 2

Instructions

BUY NOW

Personal details





STEP 3

To activate your test, you must enter your unique Test ID Code found on your Activation Card plus your Online Order Reference Code. Confirm by clicking 'Activate Now'.



STEP 5

Hold your hand under warm water for at least one minute to stimulate blood circulation then carefully dry your hand with a towel.





STEP 7

page, you will need to either create an account or login to your existing account to activate your test - then follow its



STEP 4

Open the blood collection tube and place it on a flat surface. Ensure that it is positioned upright and steady.



STEP 6

Stand up and swing your arm around in a circular motion 5-10 times, with your arm fully extended if possible. This helps stimulate blood circulation. Continue standing until you've finished taking your

sample, if possible



STEP 8

Hold the lancet horizontally (as shown). Rotate the yellow protective cover one full turn then carefully remove it. CAUTION: Do not use a lancet with a loose or missing protective cover.

V lifelabtesting

blood. Our convenient, pain-free blood

Lifelab Testing

Instructions



STEP 9

Place your hand on a stable surface. Hold the lancet (as shown in the diagram) diagonally against the tip of your selected finger. Press firmly with your thumb on the yellow button (which can only be pressed once).



STEP 11

Stand the blood collection tube upright. Place your finger diagonally at its rim, so the blood can run into it. Continue massaging your finger to produce enough blood flow. This may take several minutes.



STEP 13

After collecting your blood sample, press the swab firmly against the location of the bleeding to stem the blood flow.




STEP 15

Use the cap to securely close the tube by pressing it firmly until the cap clicks into place. Shake the tube well.



STEP 17

Slide your filled blood collection tube into the larger transport tube. Please avoid removing the absorbent felt.

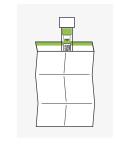

STEP 19

Place your filled blood sample transport tube into the enclosed return envelope. Close the envelope carefully - and immediately post it back to our lab. TIP: If you return vour box with your test we will recycle the box for you.



STEP 10

droplets form.

Keep vour hand horizontal at hip height (while standing at the edge of a table, for example) and massage vour finger from the palm of the hand to the fingertip until blood



STEP 12

Finish taking the sample only when the mark (30 µL) has been reached (otherwise the sample cannot be analysed). Gently tap the tube several times on the table, so that the blood runs downwards.



STEP 14

When the bleeding begins to slow, apply the plaster to the puncture



STEP 16

Remove the adhesive label from the Activation Card and affix it to the filled blood collection tube - NOT to the transport tube.



STEP 18

Seal the transport tube securely into place with its screw-top lid.



STEP 20

Place the used equipment parts in the provided clear plastic bags. Dispose of them in your household waste. NB: The test's packing and paper information inserts are all fully recyclable.



Read me -I'm your test instructions